

# ASSEMBLY INSTRUCTION


ITEM NO.:9022TG/GY-1LTM

Power Lift Chair with Massage, Heat  
and Charging Port




NOTE: Do not use power tools for assembly. Power tools increase the risk of over-tightening which leads to splitting or cracking the wood.


| NO. | DRAWING                                                                             | DESCRIPTION            | QTY  |
|-----|-------------------------------------------------------------------------------------|------------------------|------|
| A   |  | SEAT                   | 1 PC |
| B   |  | BACK REST              | 1 PC |
| C   |  | MASSAGER<br>CONTROLLER | 1 PC |
| D   |  | CHAIR<br>CONTROLLER    | 1 PC |
| E   |  | ADAPTOR                | 1 PC |

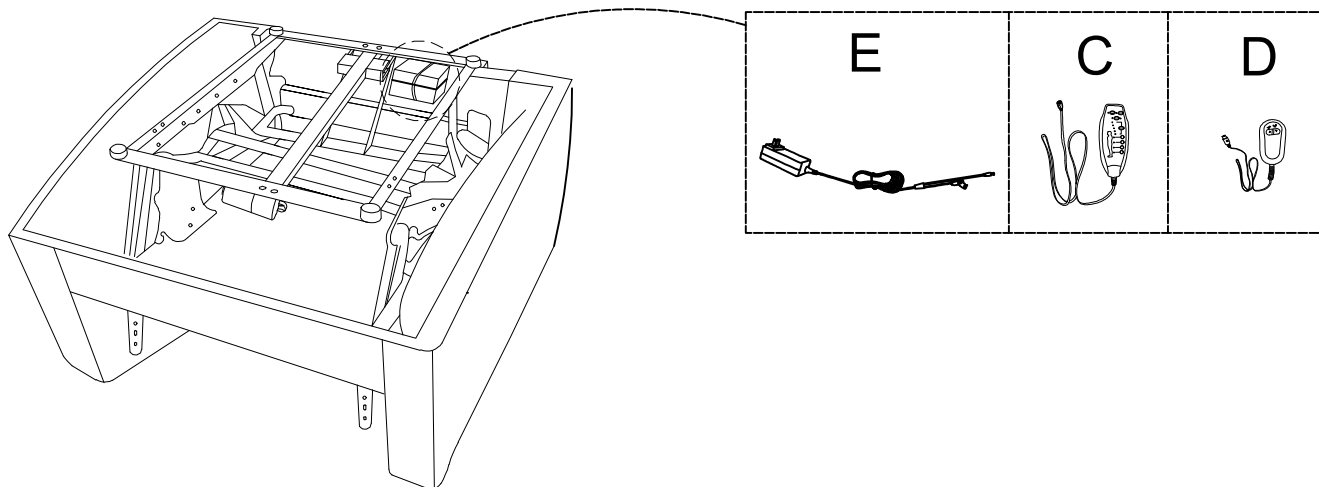


STEP 1:

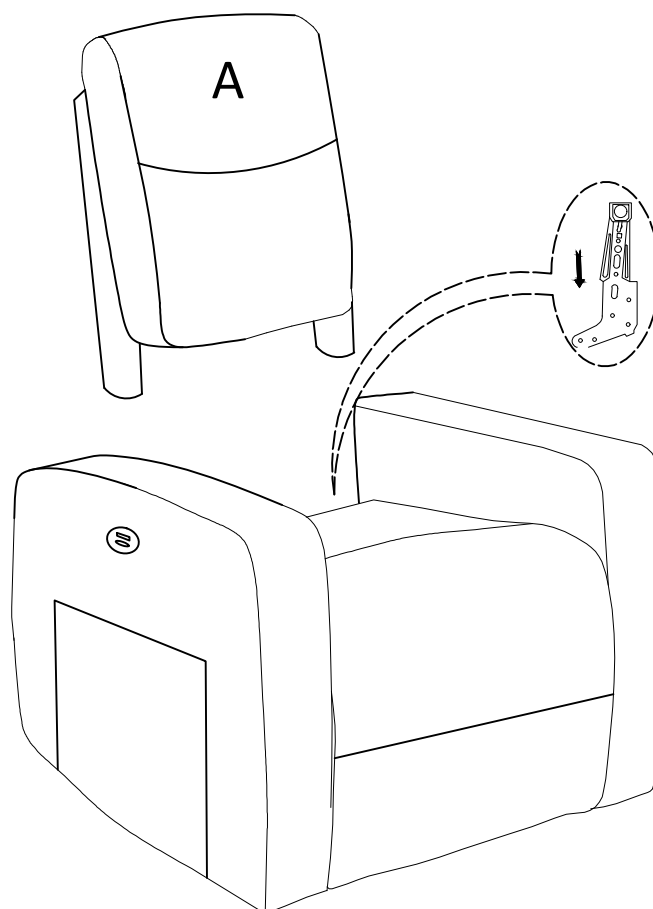


## STEP 2:


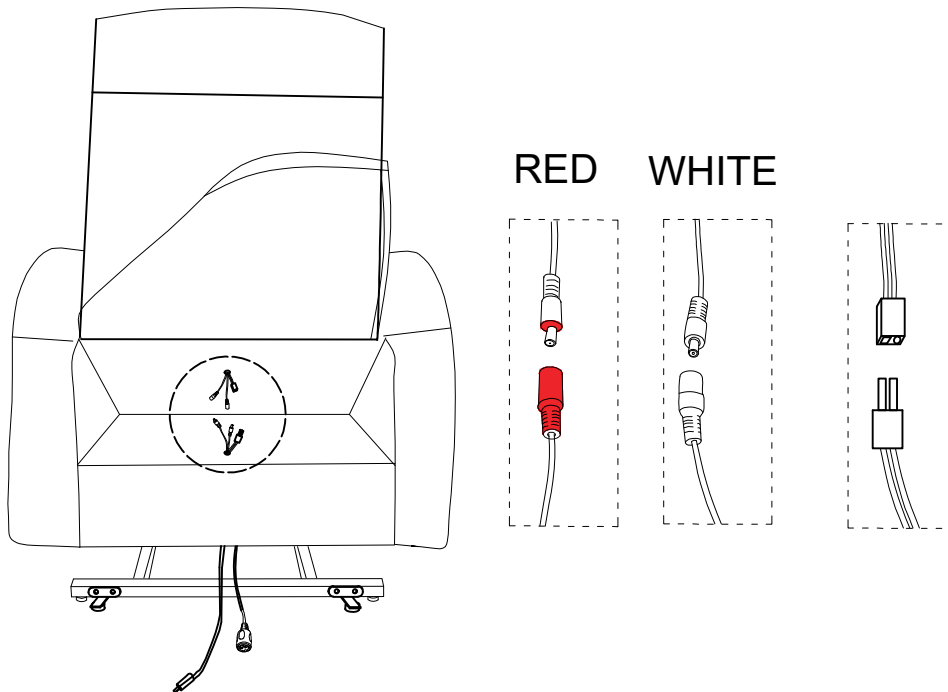





## STEP 3:

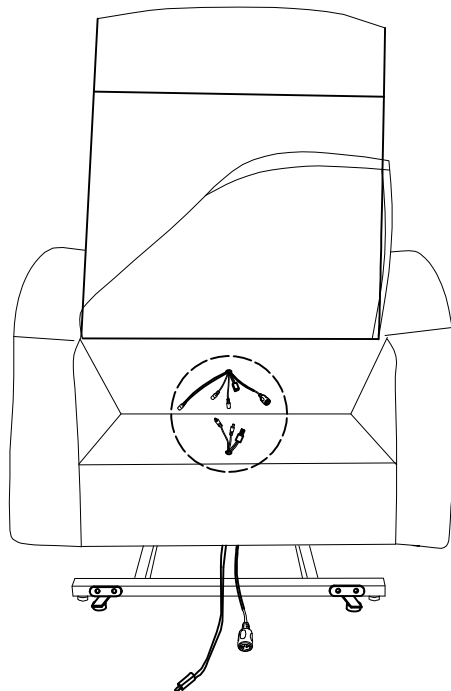


## STEP 4:

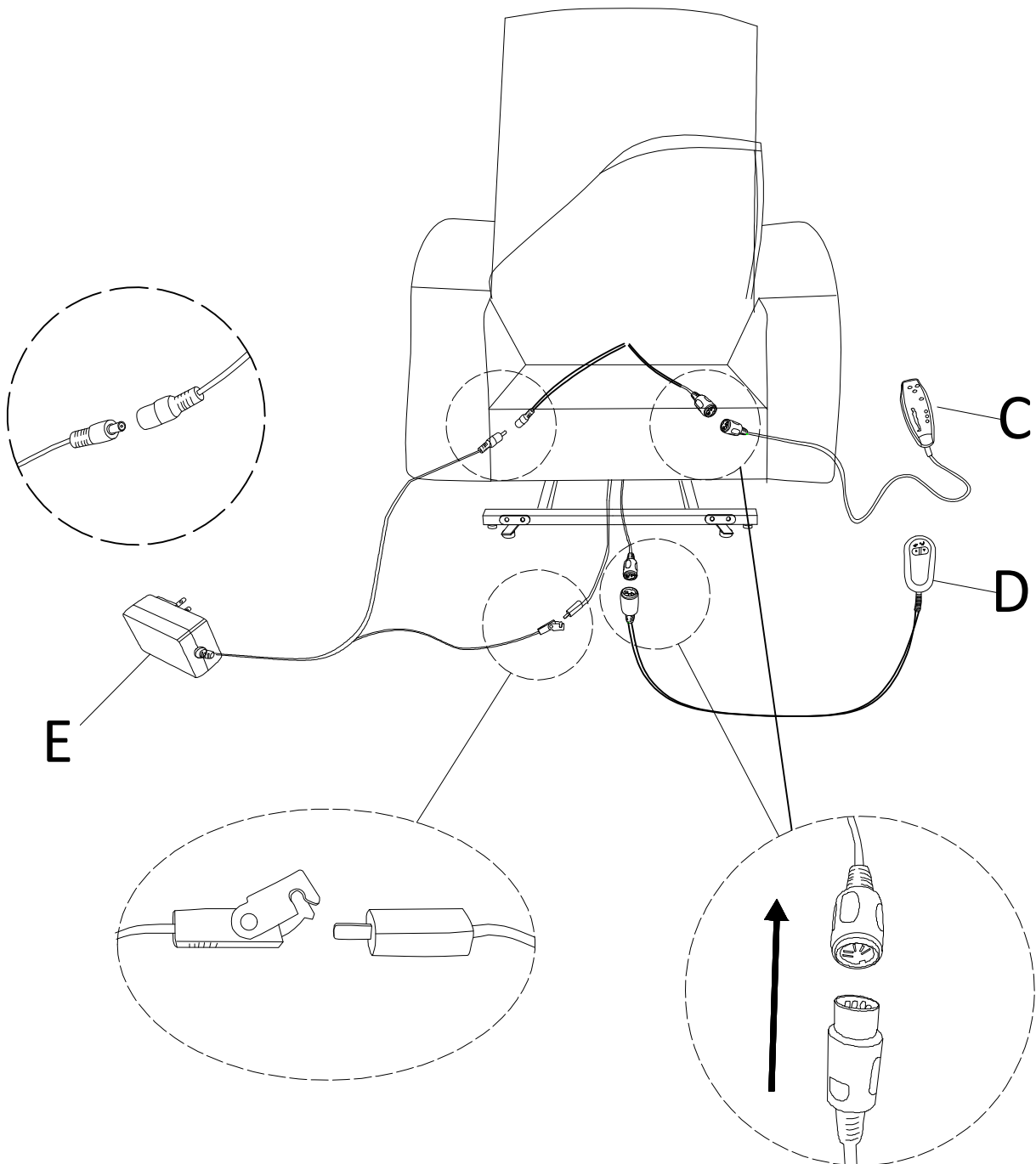

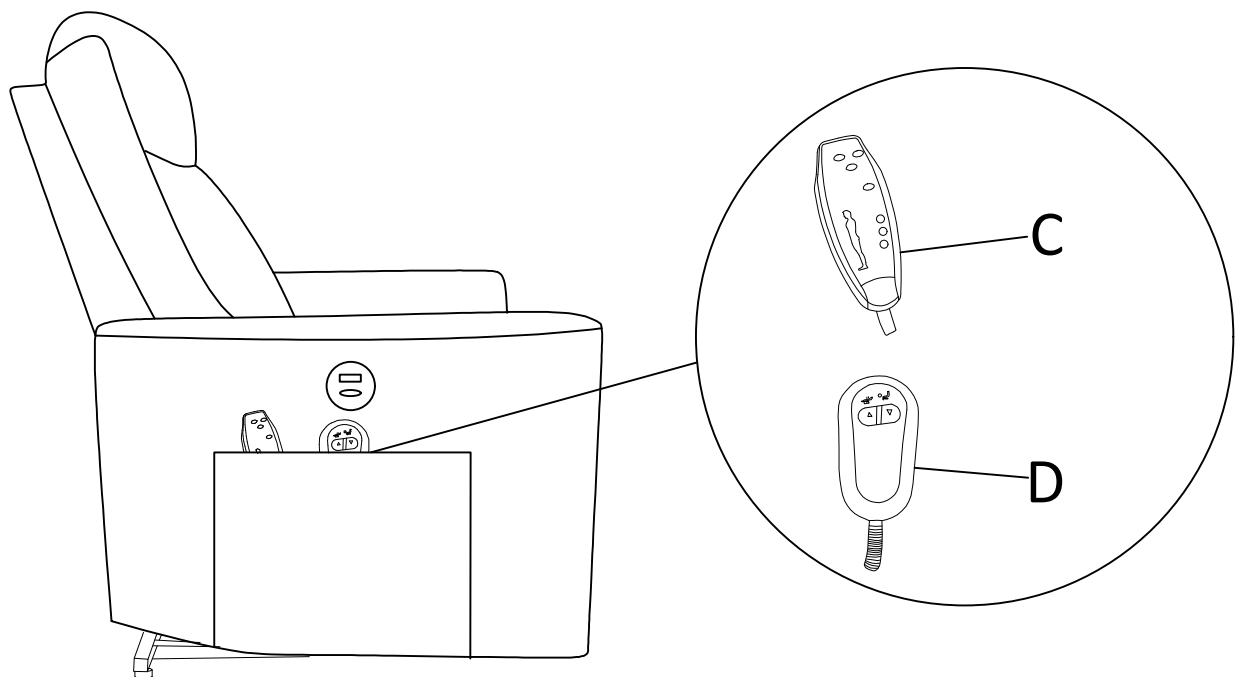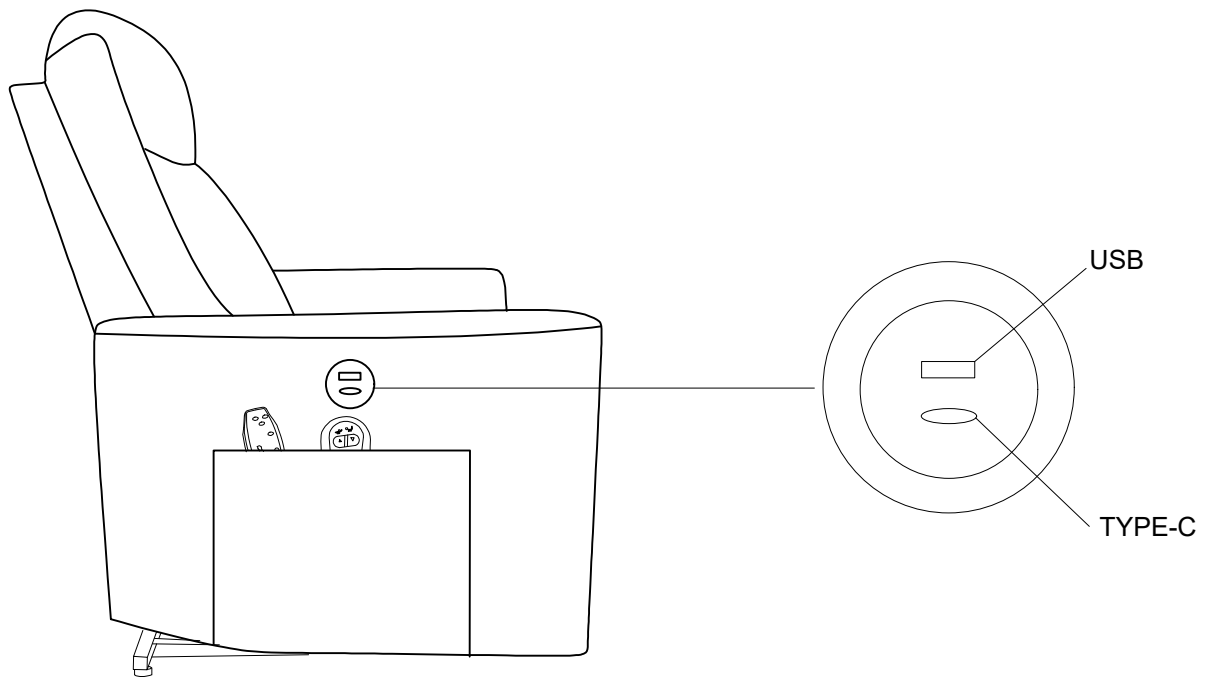

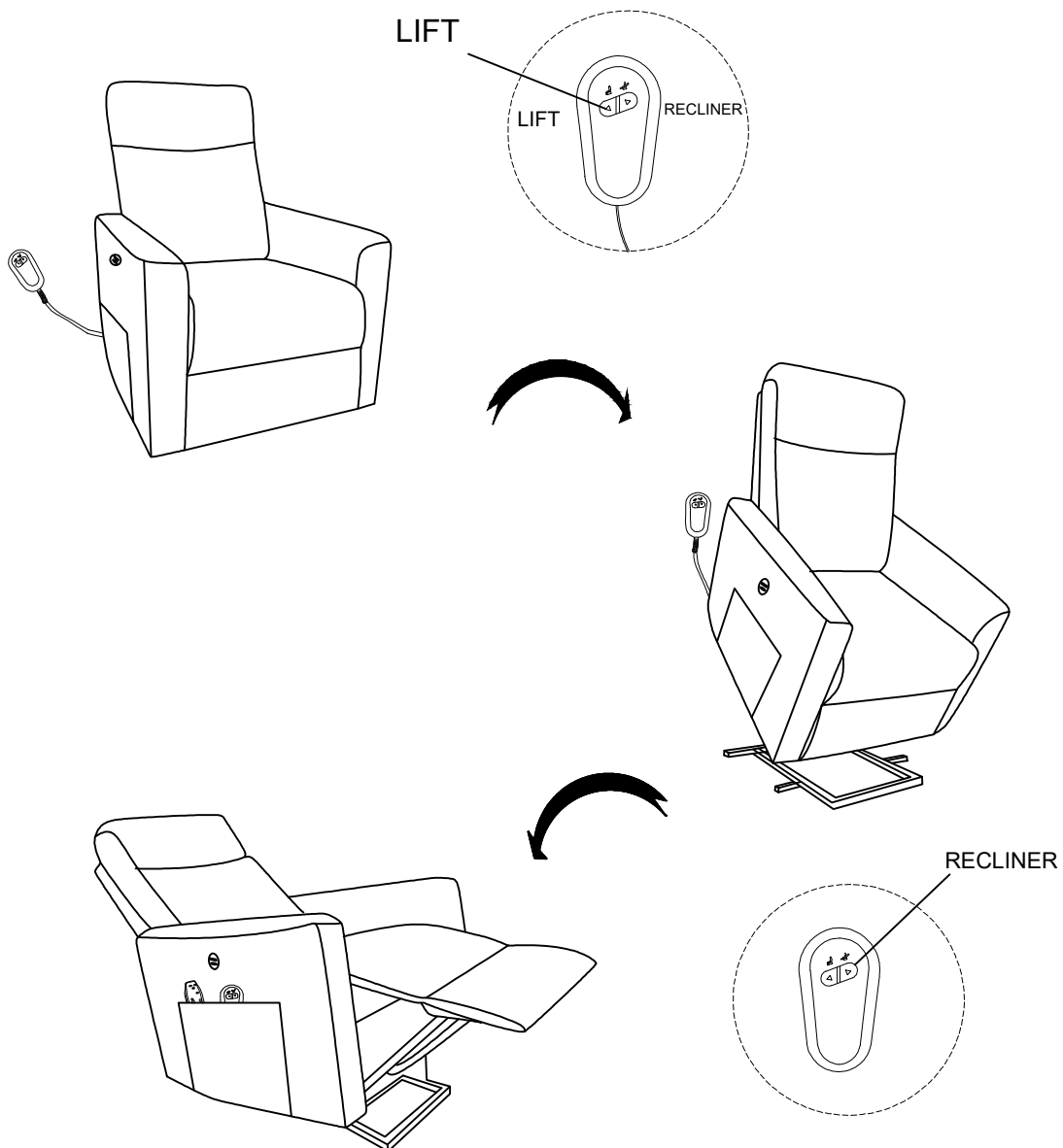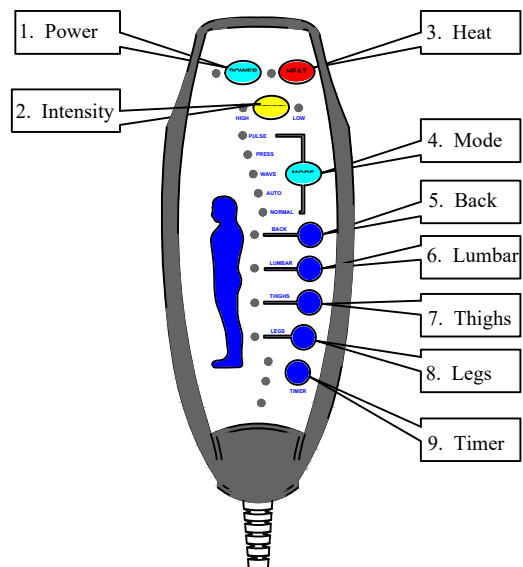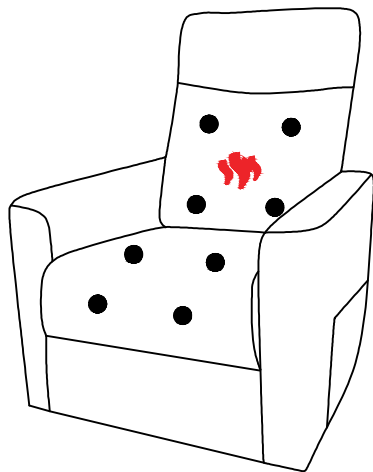

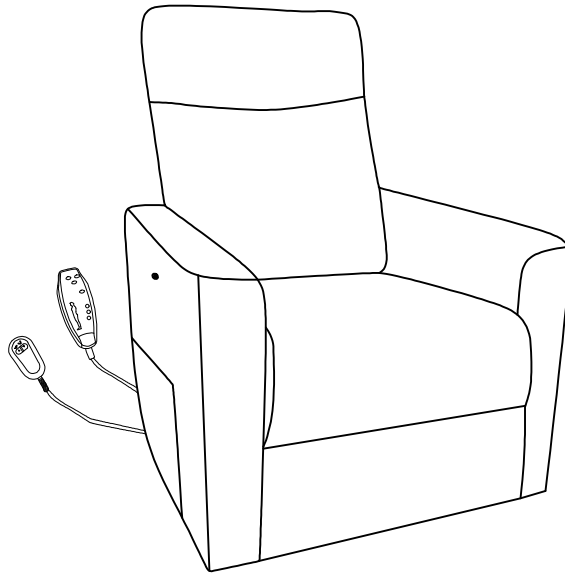

### **Care Instructions for Textured Microfiber Furniture:**

**In order to maintain the original appearance of your Textured Microfiber furniture, please follow the simple care procedures below:**

- **Avoid exposing furniture directly to sunlight or heating and air conditioning outlets. Sunlight and/or exposure to heating and air conditioning outlets will fade/damage Textured Microfiber .**
- **Vacuum or wipe with a soft cloth to remove dust.**
- **Hardware may loosen over time. Periodically check to make sure all connections are tight. Re-tighten if necessary.**




**For items used on wooden floor, hard surfaces or carpets, it is recommended to install protective pads under all legs and supports to prevent damages, scratches and discoloration.**

#### **Warning:**

**Do not allow electrical cord to be pinched between moving parts or between the frame and the floor. Do not place cord under carpet or in location where it could become worn.**

# Hand Control Instruction

## Diagram

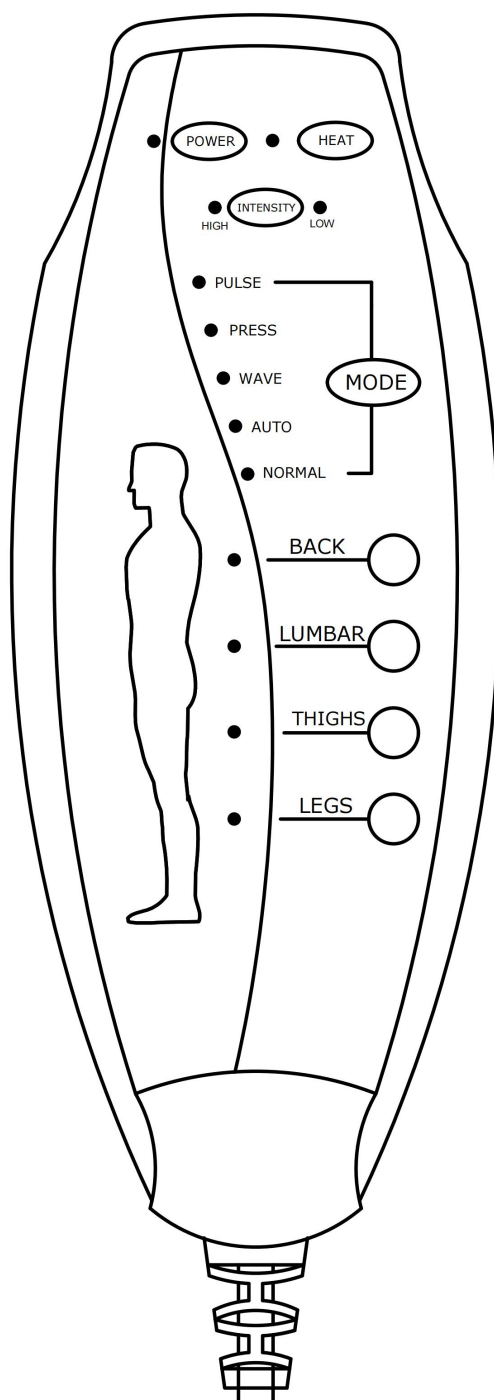

Model No. : LN-167-8JR

## Operation instructions

1. Connect the hand control cable female head of the hand controller to the hand control cable male head coming out of the sofa.
2. Connect the DC black male connector of the power adapter to the DC black female connector coming out of the sofa.

3. Press  button to turn on or off vibration massage.

4. Press  button to turn the heating on or off.

5. Press  button to switch the intensity of vibration massage, and there are two intensity levels to choose from.

6. Press  button to switch between massage modes, which can be switched in five modes.

7. Press  button to turn on or off the back vibration massage.

8. Press  button to turn on or off the lumbar vibration massage.

9. Press  button to turn on or off the thigh vibration massage.

10. Press  button to turn on or off the legs vibration massage.

## Important safety instructions

When using an electrical appliance, basic precautions should always be followed, including the following:

- 1、 This product is for use on a 12V circuit. Make sure that the product is connected to an outlet having the same configuration as the plug.
  - 2、 Always unplug the massager from the power source after use, and before cleaning.
  - 3、 Do not reach for massager if it has fallen into water. Unplug immediately.
  - 4、 An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, before putting on or taking off parts or attachments.
  - 5、 Do not use outdoors.
  - 6、 Do not carry this appliance by supplied cord as a handle.
  - 7、 Keep the cord away from heated surfaces.
  - 8、 Close supervision is necessary when this appliance is used on or near children, invalids or disabled persons.
  - 9、 This unit is not a toy. Children should not use it or play with it.
  - 10、 Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
  - 11、 Do not use near eyes or other highly sensitive areas.
  - 12、 To disconnect, push the "POWER" key to the "OFF" position, make sure all functions have stopped, then remove plug from outlet.
  - 13、 Keep cord away from heated surfaces.
  - 14、 Use this appliance only for its intended use as described in this manual.
- Do not use attachments not provided with the unit.

## Health caution

- If you have any concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should not use this product.
- If you experience pain in a muscle or joint for a prolonged period of time. Discontinue use and consult your doctor. Persistent pain could be a symptom of

a more serious condition.

## Maintenance instructions

### TO STORE:

Place unit in its box or safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric. Do not wrap power cord around the unit. Do not hang the unit by power cord.

### TO CLEAN:

- Ensure the unit is unplugged from the power source, and allow to cool before cleaning.
- Do not place in any liquid to clean. Clean fabric cover by wiping with a damp sponge, using a mild detergent or suds. Pat dry with clean, dry cloth.
- Do not use abrasive cleaners, brushes, gasoline, kerosene, glass /furniture polish, paint thinner or water hotter than 120°C to clean the unit.
- To clean the unit, make sure the massager is unplugged. Allow the unit to cool and then wipe down with a damp clean cloth. Do not use any strong chemical cleaners.